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15寸口腔内窥镜一体机  
15 inch dental monitor with oral camera system

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Features:

- Perfect combination of intraoral camera and monitor.
- With SD card memory, solve the problem photos cannot be saved.
- Monitor could take picture for patient head with built-in webcam, in order to save files.
- It could compare with quad function.
- It could manage data after connected with computer, or display multi-media file in monitor.

经销商  
Distributor

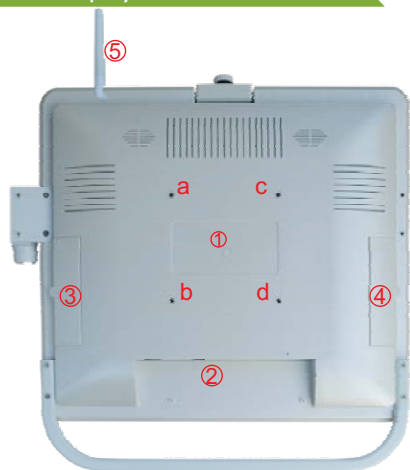
## 屏幕按钮说明 Display buttons illustration.

\* 此按键为触摸按键，使用时请用手指轻按2秒。  
\*The buttons are touch button. Please click the buttons 2 seconds with finger.

A: 电源开关 Power on/off	H: 画面回放下一张 Page Down.
B: 内窥镜与内置摄像头切换键 Toggle key for oral camera and webcam.	I: 画面回放上一张 Page Up.
C: 内窥镜与PC(VGA)信号切换键 Toggle key for oral camera and PC(VGA)signal.	J: 音量大声 Volume Up.
D: 有线/无线内窥镜切换键 Toggle key for wire/wireless oral camera.	K: 音量小声 Volume Down.
E: 单画面与四画面切换键 Toggle key for single and quad image.	L: 显示器把手 Monitor Handle.
F: 遥控器红外线接收端与电源指示灯 Remote control IR receiver and power indicator.	M: 内窥镜手柄挂架 Camera Holder.
G: 画面删除键 Delete key.	N: 显示屏内置摄像头 Monitor built-in webcam.

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
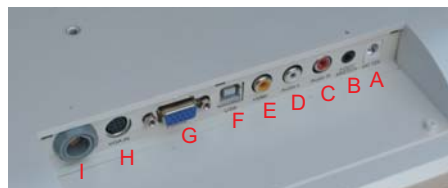
#### Monitor connecting port illustration.

A: DC 12V 3A 电源插口 DC 12V 3A power in	F: USB连接电脑影像声音插口 USB port to connect with PC video and voice
B: 脚踏开关插口 Pedal in	G: VGA 电脑信号输入 VGA port for PC signal input.
C: 视频声音右声道输入 Video voice R-CH input	H: 外接内窥镜插口 External oral camera port.
D: 视频声音左声道输入 Video voice L-CH input	I: 有线内窥镜插口 Wire oral camera port.
E: Video视频输入 Video input	

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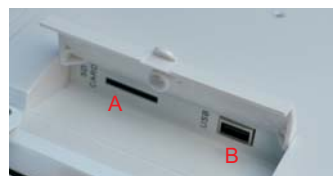#### SD card slot and external USB port.

A: SD卡槽  
SD CARD

B: USB 2.0  
(在连接电脑的情况下才有功能)  
It can be used when connected with PC

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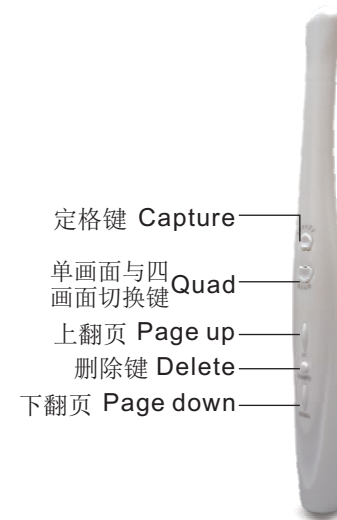
\*Camera capture button is save images into virtual memory(virtual memory could save 8pcs photos only), if you want to preserve permanently, please use remote control save key store into SD card and copy to computer.

## 无线发射器与接收器 Wireless Launcher & Receiver

无线发射器  
Wireless Launcher

无线接收器  
Wireless Receiver